

# Like nothing else. From no one else.

## Livio Edge AI | Livio AI | Livio



The industry's most complete line of 2.4 GHz rechargeables  
More options than ever before



### Custom Rechargeables

- Available in ITC/HS R & ITE R
- World's first 2.4 GHz custom rechargeables
- Up to 130/70 dB matrix
- Telecoil (ITE R only)
- Push button, VC, or dual controls
- Custom Charger

NEW



### BTE Rechargeable

- World's smallest, most powerful BTE rechargeable\*
- 130/70 dB matrix
- Telecoil
- Push button
- Thin tube compatible
- Standard and Mini Turbo Chargers



### RIC Rechargeable

- 40, 50, 60, 70 dB gain receivers
- Telecoil
- Rocker switch
- Standard and Mini Turbo Chargers



Standard Charger



Custom Charger



Mini Turbo Charger



### Thrive Hearing Control App

Delivering convenient usability, innovative new features, and robust connectivity.

- **NEW** Interactive setup and onboarding process makes it simple and easy for patients to get started and engage with the Thrive app.

An unmatched fitting experience on every level



NEW

### Live Sessions in Hearing Care Anywhere

Now offering synchronous remote programming with live video

- There is no action required in the Thrive app prior to joining a Live Session.
- Apple will stream\*\* through hearing aids; Android will use the phone's audio.

NEW

### Multiflex Tinnitus Pro

Exclusive to Livio Edge AI hearing aids

- White noise
- Audiogram-shaped **NEW**
- Custom **NEW**

Measure minimum detection and minimum masking levels to assist in setting the stimulus.

NEW

### REM Target Match

Four simple steps to achieve an automatic target match. Compatible with all products, styles and technology tiers on Muse products and forward.

For use with several REM systems:

- Otometrics Aurical
- MedRx Avant
- Inventis Trumpet

\*Starkey Competitive Benchmarking Study 2019

\*\*Although streaming direct, the program will not be Streamboost and will stream in whichever program the hearing professional has selected.

# The best just got better.

## Livio Edge AI | Livio AI | Livio

### Hearing Performance & Healthable Technology Features


**Edge Mode**  
Give your patients the edge; no need to address unique listening environments with generalized programs and memories. Edge Mode instantly conducts an AI-based analysis of the current listening environment and makes smart, immediate adjustments.

- This AI algorithm is on board the hearing aid – **no need for the patient to have their smartphone.**
- **A great choice for face masks:** instantly optimizes to compensate for any face mask type, social distance and background noise.
- Configure access of the feature within the Inspire X User Controls screen or in the Thrive app.

**Thrive Assistant**  
Patients use **voice commands** to get answers to questions, set reminders and control their hearing aids.

**Find My Phone** **NEW**  
Using their hearing aids and Thrive Assistant, wearers are able to locate their lost or misplaced phone by having it ring.

**Mask Mode** **NEW**  
New custom memory in the Thrive app **boosts speech audibility.**

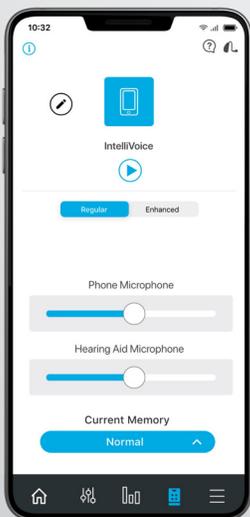

### IntelliVoice **NEW**


Thanks to the power of AI, this first-of-its-kind speech enhancement feature uses the wearer's Apple smartphone to deliver an unprecedented experience to those with a **loss of at least 50 dB HL or greater at 250 and 500 Hz.**

Available in the Advanced version of the Thrive app within the Accessories menu.

IntelliVoice features **two modes:**

- **Regular** is similar to the Live Listen function but without the lowpass filter imposed by iOS.
- **Enhanced** uses the IntelliVoice algorithm that taps into deep neural network parameters to enhance speech like never before.

### AI for All

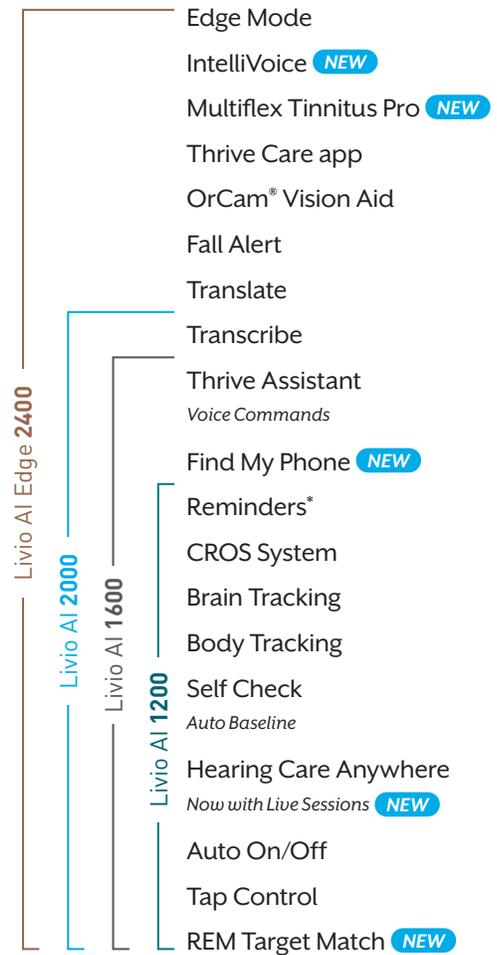

New features are accessible through a performance update.

### 2.4 GHz Accessories

Our exceptional line of accessories connects patients wirelessly to their world.



\*Reminders is available for purchase within the Thrive app for the 1200 and 1000 technology tiers.

The app may look slightly different depending on your phone and/or hearing aid model.

Starkey, Starkey logo, Inspire, Inspire logo, Livio, Hearing Care Anywhere, Thrive, Thrive logo, IntelliVoice, Thrive Care and Healthable are trademarks of Starkey Laboratories, Inc.

Apple is a trademark of Apple, Inc. registered in the U.S. and other countries. Android is a trademark of Google LLC. OrCam is a registered trademark of OrCam Technologies LTD. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks is under license. iOS is a trademark or registered trademark of Cisco in the U.S. and other countries and is used under license.